

6 in 1 Liquid Dermabrasion™ Machine



Product parameter

Product name: 6 in 1 Liquid Dermabrasion™ Machine

Product type: HYDRA5000

Power: > 350W

Display screen: 7 inches

Ultrasonic: 1.1MHZ

RF: dipolar

Weight: 8kg

Product size: 10*14*10in

Product parameter	
Product name	6 in 1 Liquid Dermabrasion™ Machine
Product type	LB245
Power	> 350W
Display screen	7 inches
Ultrasonic	1.1MHZ


Working time

D(H₂,O₂)
switch key

Waste bottle state

RF	dipolar
Weight	8kg
Product size	10*14*10in

Product structure

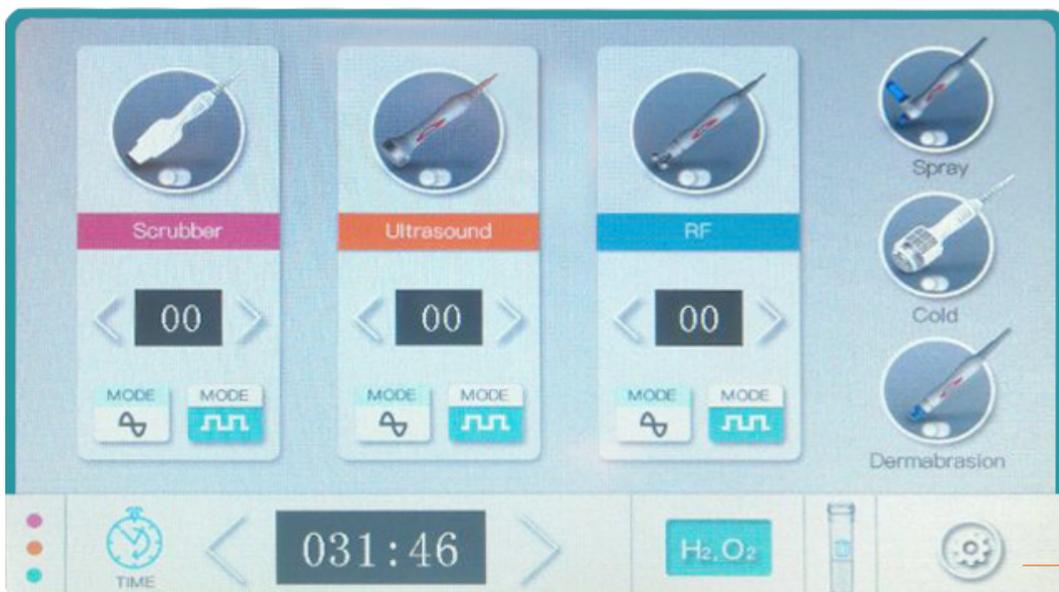



Accessories

1. **Small bubble pen:** move quickly from middle to two sides, without staying in one area too long. For sensitive skin, use the plane and vortex suction head, thick pore and abrasive suction head.



2. **Oxygen spraying pen:** evenly spray essence liquid products on skin for better absorption.




3. **Freezing probe(cold):** shrink pores, release ice force, permeate from epidermis to deep skin. Instantly make the skin pores shrink from the natural freezing mask to lock nutrients on the skin's surface.

4. **Ultrasonic probe:** ultrasonic can accelerate blood circulation and promote metabolism through the compression and extension of megahertz ultrasonic.



5. **Dipolar RF probe:** micro-current lifting. Improve eye wrinkles. Lift and tighten, to decrease bags under eyes.

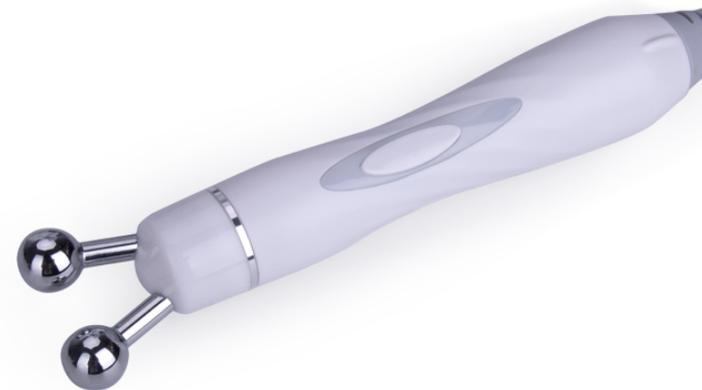

6. **High frequency scrubber:** vibration, remove dead skins, whiten skin, remove blackheads and dead skin corneum.





Function

- 1.Remove blackheads
- 2.Deep cleaning
- 3.Face lifting
- 4.Replenish nutrition
- 5.Skin whitening
- 6.Shrink pores
- 7.Skin moisturizing
- 8.Vibrate and massage

Advantages

- 1.Touch screen with quick adjusting.
- 2.Plugin design and can work continuously.
- 3.One machine with multiple functions.
- 4.Esthetically attractive design.
- 5.Non-invasive.
- 6.It is suitable for various skin types:neutral skin, dry skin, sensitive skin, oily skin and skin with acne.

Comparison

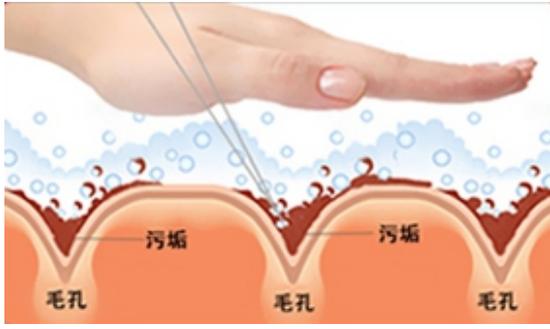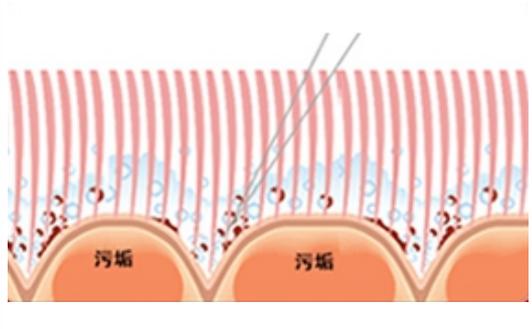Traditional hand wash	Ultra-micro small bubbles
 A cross-sectional diagram of a hand being washed. A hand is shown at the top with water and bubbles. Below the skin surface, three pores are labeled '毛孔' (pore) and each contains a dark brown substance labeled '污垢' (dirt). The dirt is only on the surface of the pores.	 A cross-sectional diagram of skin with ultra-micro small bubbles. The skin surface is covered with many small red vertical lines representing bubbles. Below the surface, two pores are labeled '毛孔' (pore) and each contains a dark brown substance labeled '污垢' (dirt). The bubbles are shown permeating the skin layers and reaching into the pores.
Cleans the skin's surface, but can't deeply clean. Also easy to cause clogged pores.	Deeply permeate skin's underlying layers. Opens pores and takes away facial grease and dirt.

Operation steps

- 1.Simple cleaning methods.
- 2.Use oxygen injection pen, spray hot for 10 minutes to open pores.
- 3.Use scrubber.
- 4.Choose one kind suction head and liquid according to personal preference to install into small bubble pen.
- 5.Use oxygen injection pen to absorb liquid.
- 6.Use dipolar RF eye probe on each eye for 5-10 minutes. Use upward lifting or in circular motions.
- 7.Use Freezing probe, apply gel to face, operate to calm skin for 5-8 minutes.
- 8.Use ultrasonic function with essence, use probe for 5 minutes.
- 9.Apply recovering face mask for 15 minutes.

Operation steps

- 1.Make-up removal and cleaning.

Scrubber function

- 2.Hot spray for 10 minutes to open pores. Then use the scrubber. Choose continuous energy and adjust scrubber speed(advice: speed level 5).



Dermabrasion suction head

- 3.Choose one suitable suction head according to skin, click dermabrasion function.
The left rotary knob can adjust water flow, middle is for adjusting liquid bottle position, right is for adjusting suction level. Adjust the rotary knob to bottle A, then adjust suction and water flow and click start.
- 4.After finishing the whole face cleaning, adjust the rotary knob to bottle C to clean water dermabrasion pipe. Then adjust water flow and suction to max. Start the

machine and press
thumb, repeat 3-4
5. Adjust rotary knob
moisture and nut



head for 5-10 seconds by thumb. After, loosen the
the cleaning.
start the machine and supplement plenty of
minutes.

6. Adjust rotary knob to bottle C and clean the machine pipe. Firstly adjust water flow and suction to max. Start the machine, press the suction head for 5-10 seconds by thumb, then loosen the thumb and repeat 8-10 times during cleaning.

7. Choose bottle D (hydrogen and oxygen water element) according to skin situation. The whole face cleaning: 10 minutes.



Oxygen spraying pen

8. Pick up spraying pen, pull the liquid essence into the little bottle of the pen. Click spraying pen function (on screen) to start machine. The whole face injection time: 15 minutes.




RF (Radio Frequency)

9. Dipolar RF eye probe: click dipolar RF function, adjust energy mode, choose continuous energy, adjust working intensity and then adjust energy to 5 and start the machine. For the eyes treatment area apply gel. Operation time is 5-10 minutes for each eye. Use the upward lifting or outward circular hand motions to operate around the eye. Also it can do the face lifting according to the skin texture.


Freezing probe

10. Click freezing cold function to start machine and smear gel on face. Circle from up to down. This can create calming and pore shrinking. Operating time is 5-8 minutes.

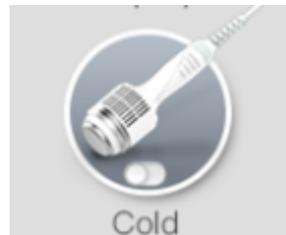

Ultrasonic

11. Click ultrasonic function, choose continuous energy, adjust energy, smear face essence in a circular motion to operate for 5 minutes on face.

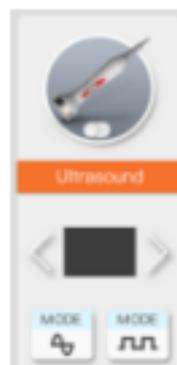

After the treatment

12. Apply facial mask for 15 minutes + facial repair gel for 15 minutes. Remove the mask and clean. Then apply skin care cream.

Attention matters

1. For seriously thin corneum people, do not use bubble suction pen, RF pen or scrubber.
2. Serious red couperose skin, do not operate RF for long periods of time.
3. Serious allergic skin, do not use this device.
4. Serious acne skin, do not use this device.

